[Note] Do not mix with G.I cement or thymol cement because not curing.
When extruding temporary cement from a mixing pad, it is not cured the unless the catalysis and base are mixed at 1 to 1.

■ Caution

1. For dental use only.
2. It is not used in patients with a history of rash and hypersensitivity to dermatitis.
3. This product is used only for oral tissue. Be careful not to get into your eyes.
4. If you contact your skin or come into eyes, wash them several times with clean water and consult a specialist if necessary.
5. This product is prohibited for use other than its intended use.
And the use of non-professionals is prohibited.
6. Observe the procedure and the precautions.
7. Keep it away from a reach of non-dental users including children, senior citizen and other.
8. Do not use after expiration date.
9. There is a risk of product damage, so don't give excessive impact.

■ Storage

1. Do not store it under conditions of high temperature or intense light.
2. Close the cap immediately after use.
3. Store in dry and cool place. (1~30°C)

■ Contents

- # 201-1 Any-Temp NE 10g x 2syringes / Mixing Pad / ECO-Tips
- # 201-2 Any-Temp NE 10g x 3syringes / Mixing Pad / ECO-Tips

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Any-Temp NE is zinc oxide non-eugenol temporary cement for cementation of temporary restoration.

■ Indications

1. Dental temporary cement.
2. Temporary cement of temporary restoration.

■ Composition

Zinc oxide 70~85%, Rosin 50~60%, Nonanoic acid 30~40%

■ Application method

1. Before the application

- ① Check the expiration date and be cautious of the package contaminated.
- ② Follow the instructions before using it.
- ③ Be sure a protection device for the respiratory before the procedure.
- ④ After use, close the cap immediately to minimize volatility.

2. Applications

- ① Extrude Any-Temp NE onto the mixing pad.
- ② Dry the prepared teeth and internal surface of the restoration.
- ③ Thoroughly mix the dispensed Any-Temp NE for 20 sec.
- ④ Apply the mixed Any-Temp NE cement to the internal surface of the temporary restoration.

[Working time : 1 min, Setting time : 3 min.]

- ⑤ Firmly seat the restoration in the mouth.
 - ⑥ Trim away excess cement after the material has set.
- [Approximately 5 min. from start of mix.]

■ Contraindications

1. Do not use Any-Temp NE to a patient having allergy and inflammation against rosin.
2. Tip reuse is prohibited, as it may cause cross-infection.