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MEDICLUS CO., LTD.

No.1210, 134, Gongdan-ro, Heungdeok-gu, Cheongju-si,

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Canal-Seek Root canal orifices indicator.

Canal-Seek is intended for use during endodontic treatment to identify the root canal orifices. Blue dye contained in the product invades into the root canal orifices and dyes them blue which makes it easier to find the main root canal orifices and detect the cracks.

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**■ Contre-indications**

Canal-Seek Root canal orifices indicator. Canal-Seek is intended for use during endodontic treatment to identify the root canal orifices. Blue dye contained in the product invades into the root canal orifices and dyes them blue which makes it easier to find the main root canal orifices and detect the cracks.

**■ Indications**

Disclosing caries dentin in cavity preparations.

**■ Composition**

- ① Methylene blue 35~40%
- ② Propylene glycol 10~30%
- ③ Water Ph. Eur 10~30%

**■ Application method**

## 1. Before the application

- ① Check the expiration date and be cautious of the package contaminated.
- ② Follow the instructions before using it.
- ③ Be sure a protection device for the respiratory before the procedure.
- ④ After use, close the cap immediately to minimize volatility.

## 2. Applications

- ① Before the treatment immerse attached applicator in spirit (ethyl alcohol) and then place it onto the syringe.
- ② Remove dentine where the root canal orifices are expected, insert 1~3 drops of preparation into the bottom of the tooth cavity, wait 5~10 seconds, next rinse and dry the tooth cavity.
- ③ Points intensive dyed in blue indicate the root canal orifices.
- ④ After use withdraw the syringe piston to prevent overflow of preparation, screw off applicator and then capped into the syringe with cap
- ⑤ Product intended for multiple use except the applicator which is for single use only.
- ⑥ Multiple use of applicator may cause the risk of re- infection.

**■ Contraindications**

1. Do not use Canal-Seek to a patient having allergy and inflammation against methylene blue.
2. Reuse of the enclosed tip can cause cross-infection, so re-use of the tip is prohibited.

**■ Caution**

1. For dental use only.
2. It is not used in patients with a history of rash and hypersensitivity to dermatitis.
3. This product is used only for oral tissue. Be careful not to get into your eyes.
4. If you contact your skin or come into eyes, wash them several times with clean water and consult a specialist if necessary.
5. This product is prohibited for use other than its intended use. And the use of non-professionals is prohibited.
6. Observe the procedure and the precautions.
7. Keep it away from a reach of non-dental users including children, senior citizen and other.
8. Do not use after expiration date.
9. There is a risk of product damage, so don't give excessive impact.
10. Use of this product other than components is prohibited.

**■ Storage**

1. Store in a cool place. (1~30°C)
2. Avoid exposure to direct sunlight.

**■ Contents**

# 507 Canal-Seek 3ml x 1 syringe / Brush tips

**■ Shelf-life**

2 years from the date of manufacture